
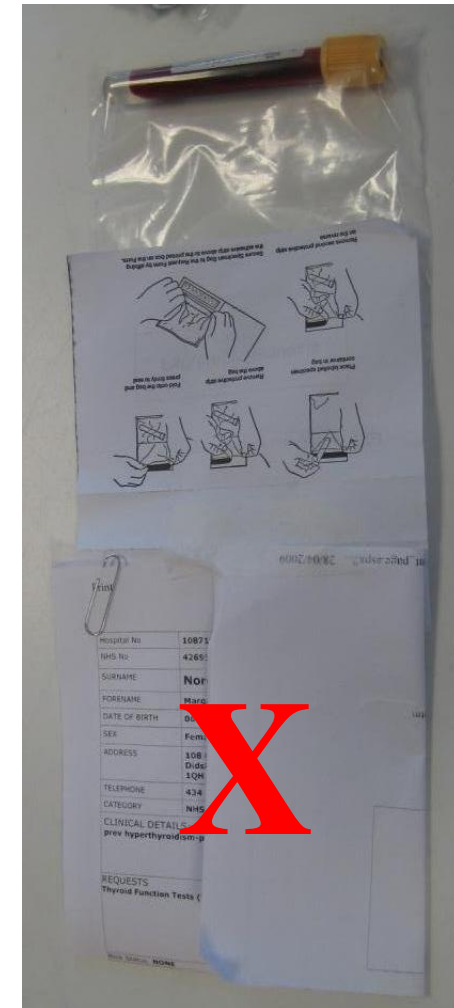


*** ANGLIA ICE – TIPS TO STOP DELAYS IN RESULTS ***

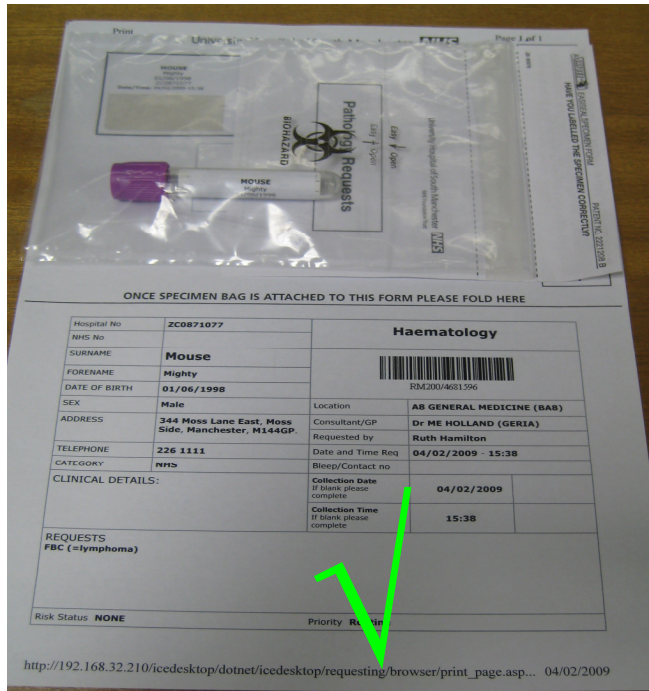

- USE ONE ICE BAG PER ICE FORM
- SEAL SAMPLES INSIDE BAG
- STICK BAG TO TOP HALF OF FORM
- FOLD AND SEND TO LAB
- DO NOT WRITE EXTRA TESTS ON FORMS, EDIT ICE & REPRINT

DO NOT PUT FORM IN BAG WITH SAMPLES

DO NOT PUT SAMPLES FROM 2 OR MORE FORMS IN 1 BAG

NO STAPLES, TAPE OR PAPERCLIPS

Remember **SAFETY (Sealed bag for samples only) ** QUALITY (Fast turnaround time) ** PATIENCE EXPERIENCE (No repeat samples)***